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
- (A) The distance from the unit varies and must be fixed together with the customer.
- (B) Mounting holes. Don't lay any lines in this area.
- (C) The dashed line shows the maximum extension length of the headrest (12 cm) and the end of the continuous backrest.
- (D) Position of the ULTRADENT ceiling lamp. **Important:** This position may differ for other makes.

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Scale 1 : 20
Scale drawing
All dimensions in mm

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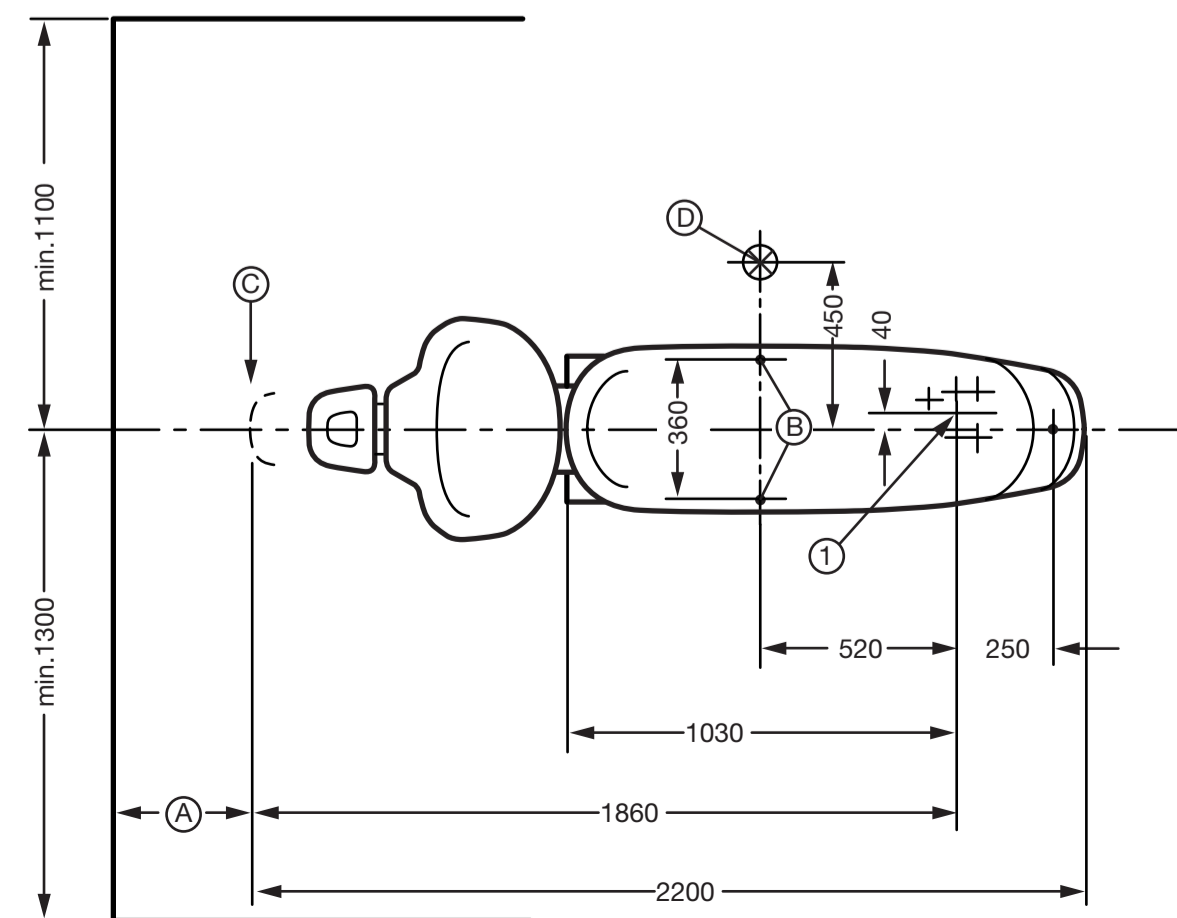Extended working position with horizontal backrest and typical height (approx. 65 cm from the floor to the bottom edge of the backrest)

Anforderungen Absaugung:	
Absaugunterdruck am Geräteeingang/ Bodenanschluß 3:	160 mbar
Staudruck:	160 mbar
Saugleistung am Geräteeingang/ Bodenanschluß 3:	900-1000 NI/min
Requirements for suction:	
Negative suction pressure at unit input/floor connection 3, banking-up pressure:	160 mbar
Suction power at unit input/floor connection 3:	900-1000 NI/min

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Requirements for water supply:	
Water quality at angle valve:	Drinking water quality
Water hardness:	8,4 - 12° dH 1,50 - 2,14 mmol/l
pH:	7,2 - 7,8
Filtration:	0,08 mm/80 µm
Water pressure:	2 - 5 bar

Please note:
Failure to comply with the above-mentioned values can lead to malfunctions. Scaling problems may occur if the maximum water hardness is exceeded. A water softening system that works according to the ion exchange principle should be installed. Algal deposits may be formed if the water hardness is less than the minimum value.

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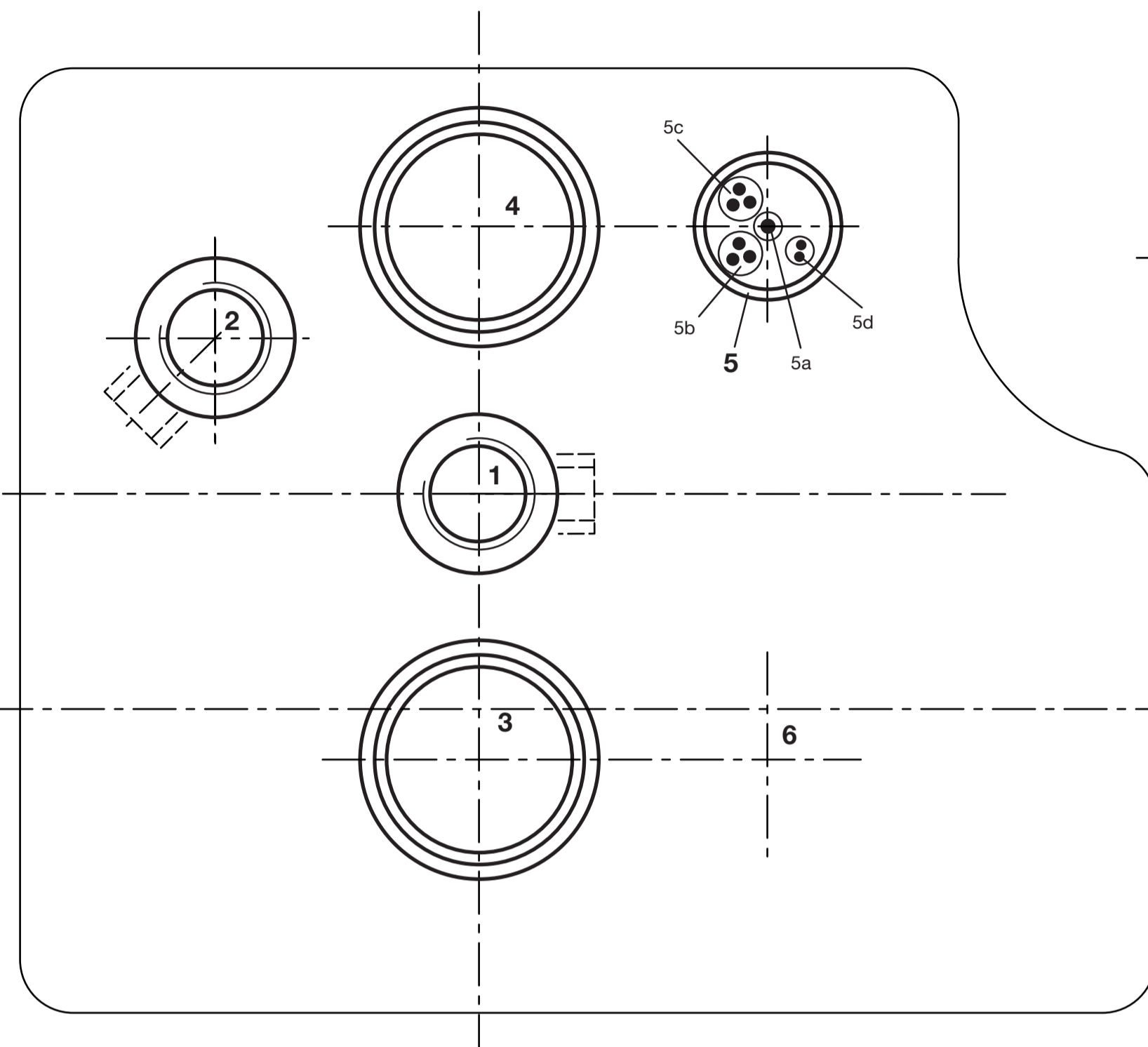
IMPORTANT!
These units may only be connected to installations that comply with VDE regulations 0100 and 0107.

The plumbing must be carried out in accordance with the technical rules described in DIN 1988, Part 4 and DIN EN 1717, Protection against pollution of potable water installations by backflow. Prior to connecting the dental unit to the general drinking water supply, in other words, you should make sure it incorporates the required measures for the protection of potable water (free outlet) or is equipped with an internal water supply in the form of an exchangeable pressurized water bottle. If not, a suitable protective device must be installed in the water supply line by the operator.

Please observe the suction machine manufacturer's instructions with regard to suction lines.

The air used to drive the turbine, cool the drill drives and operate the cooling spray must be oil-free, dry, and meet the strictest hygiene standards. Permissible air pressure: 6 - 8 bar.

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
LEGENDE:

- 1 Water inlet (with mains angle valve for 10 Ø pipe connection and fine filter)
- 2 Compressed air (with mains angle valve for 10 Ø pipe connection and fine filter)
- 3 Suction line or discharged air DN 40 HT-PP, DIN 19560
- 4 Water drain DN 40 HT-PP, DIN 19560
- 5 Insulating tube PG 23 or PG 29 or DN 40 HT-PP, DIN 19560
- 5a Equipotential bonding conductor, min. 4 mm²
- 5b Power cable 3 x 1,5 mm², 16A
- 5c Control cable 3 x 1,5 mm², for suction machine
- 5d Cable 2 x 0,75 mm², for bell, door opener, etc. Items c, d, and e only if required
- 6 Space for ducting, for connection to other units or parts of other units
- 7 Mounting holes 9 Ø Don't lay any lines in this area!

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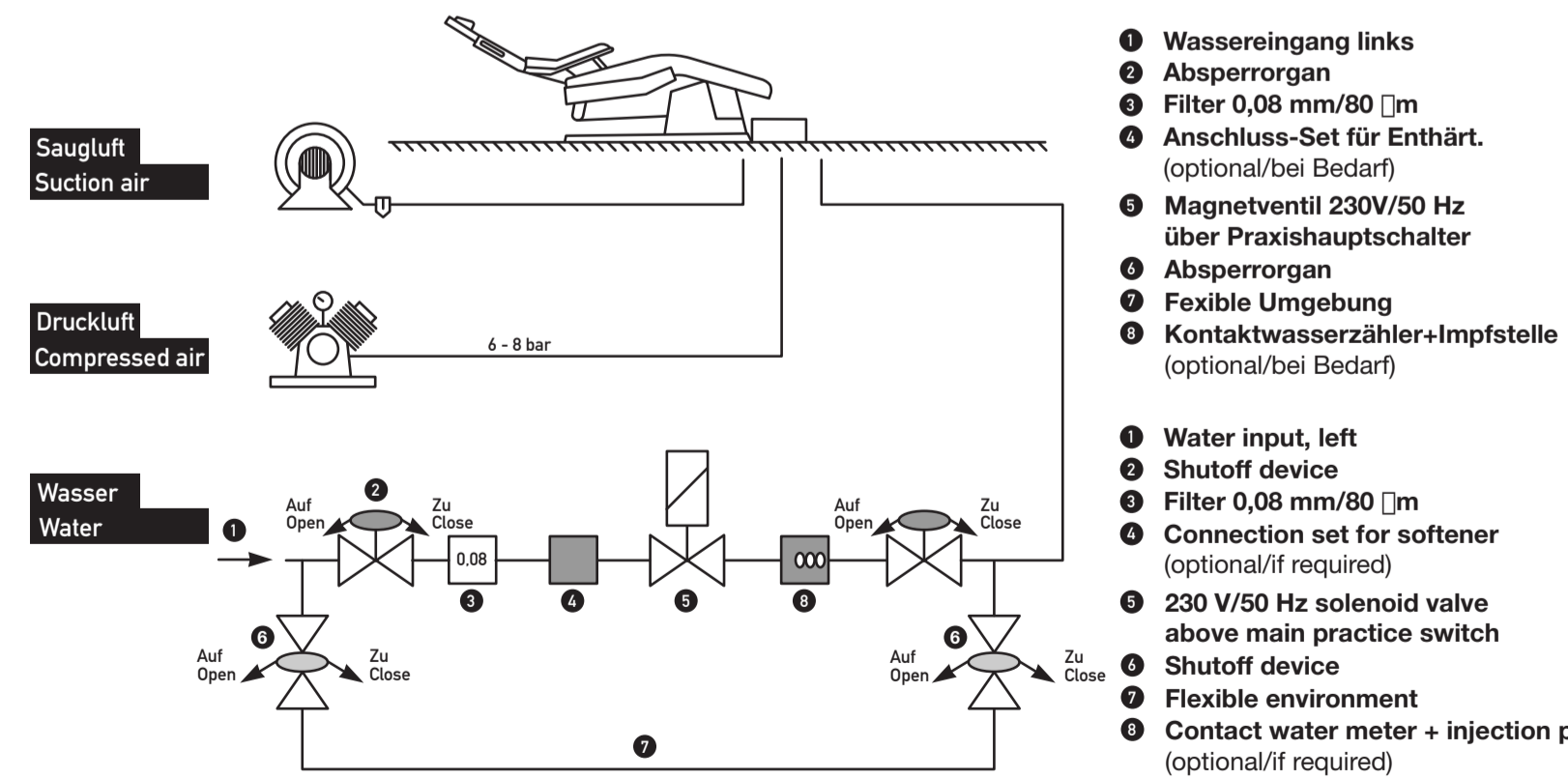- 1 Water input, left
- 2 Shutoff device
- 3 Filter 0,08 mm/80 µm (optional/if required)
- 4 Connection set for softener (optional/if required)
- 5 230 V/50 Hz solenoid valve above main practice switch
- 6 Shutoff device
- 7 Flexible environment
- 8 Contact water meter + injection point (optional/if required)

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